




5. Prepare for Injection

- Remove outer needle cover and put it aside
- Remove inner needle cover

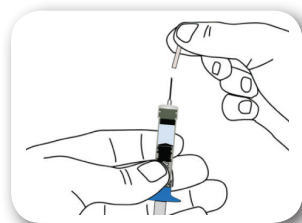


6. Inject Your GENOTROPIN

- Wipe site with alcohol swab
- Pinch a fold of skin at site, hold pen straight up, and push GENOTROPIN MiniQuick firmly into fold
- Push the plunger rod in all the way. Wait 5 seconds, then pull the needle straight out

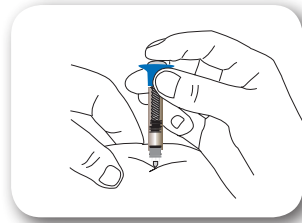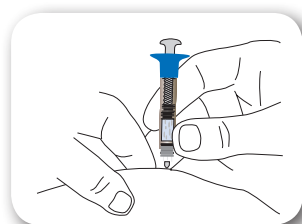

7. Finish Up

- Carefully replace outer needle cap
- Safely dispose of GENOTROPIN MiniQuick as your healthcare provider instructed

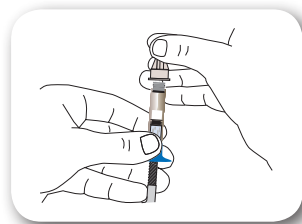

Please see full Prescribing Information and full Instructions for Use in pocket.


Growth Hormone Delivery Device containing Genotropin®
Lyophilized Powder (somatropin) for injection



Have questions about your
GENOTROPIN MiniQuick®?
Call the Pfizer Bridge Program®



or visit
www.Genotropin.com

IMPORTANT SAFETY INFORMATION

Growth hormone should not be used to increase height in children after the growth plates have closed.

Growth hormone should not be used in patients with diabetes who have certain types of diabetic retinopathy (eye problems).

Growth hormone should not be used in patients who have been recently diagnosed with cancer, with cancer, or who are being treated for cancer. Growth hormone deficiency can be caused by brain tumors. So, the presence of these brain tumors should be ruled out before treatment is started. Growth hormone should not be used if it is shown that a previous brain tumor has come back or is getting larger.

Growth hormone should not be used in patients who are critically ill because of surgery, trauma, or respiratory failure.

Growth hormone should not be used in children with Prader-Willi syndrome who are very overweight or have severe breathing problems.

GENOTROPIN should not be used by patients who have had an allergy or bad reaction to somatropin or any of the other ingredients in GENOTROPIN. In the event of an allergic reaction, seek prompt medical attention.

Rx only

You are encouraged to report negative side effects of prescription drugs to the FDA. Visit www.fda.gov/medwatch or call 1-800-FDA-1088.

Please see full Prescribing Information and full Instructions for Use in pocket.

PP-GEN-USA-0576

© 2018 Pfizer Inc.

All rights reserved.


Growth Hormone Delivery Device containing Genotropin®
Lyophilized Powder (somatropin) for injection

Printed in USA/April 2018

 10%
TOTAL RECOVERED FIBER






INSTRUCTIONS FOR USING YOUR GENOTROPIN MINIQUICK®

Use with full Instructions for Use
for the GENOTROPIN MiniQuick (in pocket)

Important Note: Please read the full GENOTROPIN MiniQuick Instructions for Use before you use your device. If there is anything you do not understand or cannot do, please call your healthcare provider or Pfizer Bridge Program® at **1-800-645-1280**.

If you have any questions about your dose or your treatment with GENOTROPIN®, please contact your healthcare provider.

Components of the GENOTROPIN MiniQuick

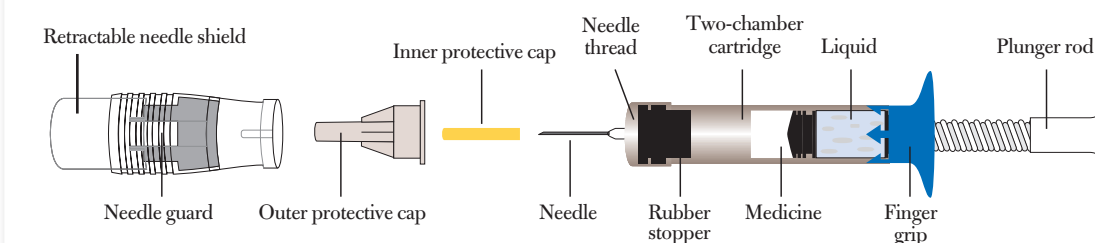

GENOTROPIN MiniQuick is a device used to mix and inject a single dose of medicine. Each GENOTROPIN MiniQuick comes loaded with a 2-chamber cartridge of medicine. GENOTROPIN MiniQuick comes in 10 different doses. **Be sure you have the dose your doctor prescribed.**


Growth Hormone Delivery Device containing Genotropin®
Lyophilized Powder (somatropin) for injection

A healthcare provider may help you with the first injection. He or she can also train you on how to inject GENOTROPIN.

Before reconstitution, GENOTROPIN MiniQuick can be stored without refrigeration at room temperature (77°F or less) for up to 3 months.

Please see full Prescribing Information and full Instructions for Use in pocket.




**Use with full Instructions for Use
for the GENOTROPIN MiniQuick® (in pocket)**

First, wash your hands with soap and water. Then, follow the steps below.


1. Prepare the Needle

- Wipe rubber stopper with alcohol swab
- Peel seal from back of needle. Leave inner and outer needle covers on

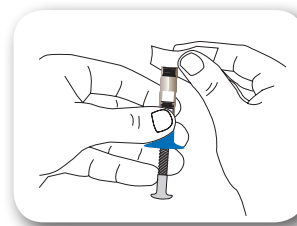


2. Attach the Needle

- Attach needle to GENOTROPIN MiniQuick by pushing it straight down and turning it to the right as far as it will go

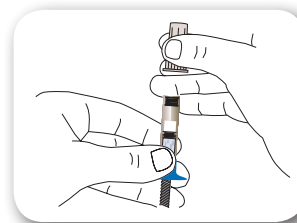

3. Mix the Dose


- Hold the GENOTROPIN MiniQuick upright, needle up
- Turn plunger to the right (clockwise) as far as it will go. This mixes your medicine
- **Do not shake** the GENOTROPIN MiniQuick. Shaking may cause the growth hormone to not work as well
- Make sure the solution is clear. If not, **DO NOT INJECT IT; call 1-800-645-1280**

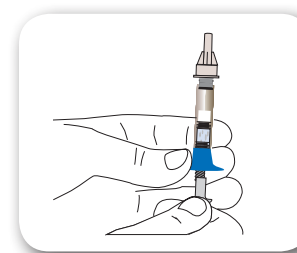

4. Select Injection Site

- Choose a different site each day

The boxes mark places where the injection can be given

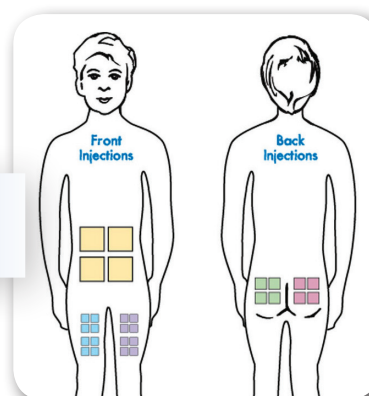

A healthcare provider may help you with the first injection. He or she can also train you on how to inject GENOTROPIN.

GENOTROPIN cartridges contain m-Cresol and should not be used by patients allergic to it.

Use a different place on the body each day for growth hormone injections. This can help to prevent skin problems such as lumpiness or soreness.